lunette®

User manual

Languages

ARABIC - CHINESE - HRVATSKI - ČEŠTINA - DANSK - DEUTSCH - EESTI - ENGLISH
ESPAÑOL - PERSIAN - SUOMI - FRANÇAIS - MAGYAR - ITALIANO - JAPANESE - KOREAN
LIETUVIŲ - NEDERLANDS - NORSK - POLSKI - PORTUGUÊS - РУССКИЙ - SVENSKA
SLOVENŠČINA

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How does it work?

The Lunette Menstrual Cup fits nice and snug, held in position by the seal formed by the walls of the vagina and the vaginal muscles. Your interior is superior and does all the work – you'll barely feel a thing!

The cup is placed entirely inside the lower part of the vagina, just behind the pubic bone below the cervix.

Vaginas are tilted backwards, so guiding the cup towards the small of your back, moving it up and down will help you find the correct and comfortable position.

Rotate the cup to check that it has fully opened and doesn't leak. Your cervix may move during menstruation, so inserting the cup requires practice and knowledge of your own anatomy.

Before first use

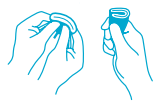
- 1. GET TO KNOW YOUR CUP**
Please read this guide carefully before using your Lunette Cup for the first time. Save it with the packaging for future reference.
- 2. WASH HANDS**
To avoid spreading any germs, wash your hands carefully.
- 3. WASH CUP**
Wash your Lunette Cup with **Lunette Feelbetter Cup Cleanser** and rinse.
- 4. BOIL CUP**
Boil your Lunette Cup in a large pot of water for 20 minutes. Make sure there is enough water so that the cup does not get burned.

Visit [instructions.lunette.com](https://www.lunette.com) for the user instructions in your native language.

Cup use wisdom

- Many like to remove Lunette Cup while straddling the toilet bowl or squatting which helps the vagina open and the cup descend closer to the opening of the vagina.
- To remove Lunette Cup, pinch the ridged cup bottom to break the seal. Don't remove the cup by pulling the stem, it will make the seal stronger, which can cause discomfort and break the stem.
- No need to remove Lunette Cup when urinating, but after defecation you may need to check that the cup is still properly in place. Wash your hands carefully!
- Occasionally Lunette Cup sits so far inside the vagina that it's tricky to reach the cup (this can happen during the night). The cup can not get lost in the vagina – it will come down by force of gravity. Wait at least half an hour, remain upright to allow it to settle and try again. Please do not panic – tensing your pelvic muscles will move the cup further away from the vaginal opening.

ATTENTION! If you have gynecological problems, discuss using Lunette Cup with your doctor.

FOLD + HOLD

WASH your hands first. Get comfy: you can insert Lunette cup while sitting, standing or squatting. Spreading your legs will help with a successful and comfortable insertion. Relax.

FOLD the cup in on itself to make flat, then in half to form a C shape.

You can also try different folding methods to find the best one for you:

Punch down fold 7 fold

- Store your Lunette Cup in its own pouch, safe from dirt and cuts, protected from UV-light, properly ventilated, and not in a plastic bag or container.
- In case of discomfort due to the stem length, you may trim it with scissors.
- Lunette Cup is a medical device, registered with, e.g. FDA.

Cautions

- Use Lunette Cup only during menstruation.
- Lunette Cup is a personal hygiene product, do not share the cup.
- Lunette Cup is not sterile, therefore it must be boiled before the first use.
- Replace Lunette Cup when any signs of damage e.g. cuts or any deterioration are observed, or when it has discolored or stained due to insufficient cleaning.
- The stem is not meant for pulling and might break if pulled.

INSERT

Keep Lunette Cup rolled up and guide it rim-first into the vagina. Rotate the cup and then check that the cup has fully opened by sliding a clean finger up to the cup bottom and feel it - it should be round.

Lunette Cup can be used any time during menstruation, from heavy to light flow days. You can insert it in anticipation of the day your period starts!

NEED MORE SUPPORT?

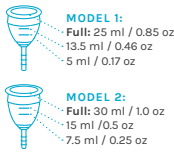
Our seriously cool Cupbot is on standby to help you with any questions, anytime: www.lunette.com

- Lunette Cup should sit in the lower vagina as a receptacle for menstrual bleeding. If the cervix is obstructed, you could experience an unpleasant feeling or cramping.
- Lunette Cup is not a contraceptive device and it cannot be used during sexual intercourse.
- Lunette Cup does not protect users from sexually transmitted diseases.
- Remember to carefully wash your hands before and after handling Lunette Cup.
- Don't use Lunette Cup for more than 12 hours at a time. Please note that local restrictions may apply: e.g. France (ANSES): 6-8 hours and no overnight use.
- Remove Lunette Cup, discontinue use and contact a doctor** if you experience symptoms such as general pain, burning, irritation, inflammation in the genital area, discomfort during urination, rash, allergic reaction (due to e.g. rare silicone hypersensitivity), or kidney-related back pain, or any adverse effects.
- Lunette Cup should not be used for anything other than menstrual bleeding, e.g. post-natal bleeding, because of a higher risk of infection.

WEAR + LEARN

Lunette Cup can be used for up to 12 hours, so it's safe to wear overnight! You may have to empty your cup more often, e.g. 3-4 times a day, depending on your flow. The measuring lines on the cup help monitor your flow and easily learn your rhythm.

* Please note that local restrictions may apply: e.g. France (ANSES): 6-8 hours and no overnight use.

- Keep Lunette Cup away from children and pets.
- Using Lunette Cup with an IUD or contraceptive ring should not be a problem, however, we recommend consulting your doctor before use. Check the correct placement of the IUD regularly - if you are unsure of the its placement or its strings, you should check with your doctor. Pay attention to the right cup removal technique and release the pressure. Please note that an IUD may partially or completely be displaced from the uterus for various reasons.
- When Lunette Cup is damaged or at the end of its service life, discard with household waste.

REMOVE + EMPTY

Wash your hands and relax your muscles. Push Lunette Cup lower using your pelvic floor muscles. Grasp the bottom of the cup with your forefinger and thumb. To break seal, squeeze the bottom ridged part of the cup, until you feel the suction release.

Rock it back and forth, fold the cup a bit and gently ease it out.

Do not pull the stem of the cup. It will make the seal stronger, causing discomfort and breaking of the stem.

Tip contents into the toilet.

TOXIC SHOCK SYNDROME (TSS)

Is an infection caused by bacteria entering through wounds or mucous membrane. **Symptoms include:** sudden high fever, sore throat, vomiting, diarrhea, dizziness, a rash resembling sunburn, muscle aches, fainting or blackouts. TSS is an extremely rare, potentially fatal disease occurring in all genders and ages. Early recognition and cure is vital, so if you have some of the symptoms mentioned above, remove Lunette Cup immediately, contact your doctor and tell them about the possibility of TSS.

DISTRIBUTED BY

Lune North America Inc.
Nashville, TN, USA, www.lunette.com/US

Ecohealth Pty Ltd
Australia, www.lunette.com/AU

PRODUCED BY

Lune Group Oy Ltd
Kopsamontie 138, 35540 Juupajoki, Finland

v2.0 | 6438458130002 | EN

CLEAN + SANITIZE

Lunette Cup should be cleaned before and after your periods, and after emptying. To avoid discoloration, first rinse the cup with cool water, then wash under hot water. Use **Lunette Feelbetter Cup Cleanser**, designed specifically for silicone cups. After your period sanitize the cup by boiling it in a large pot of water for 5-10 minutes.

If you are out and about, you can use **Lunette Cupwipes** to clean your cup.

ATTENTION! Strong detergents can leave residue on the cup causing irritation to your vaginal mucous membranes.

lunette
MENSTRUAL CUP

Your vagina's new best friend

ENGLISH

lunette

Remember that practice makes perfect.

WE ARE HERE FOR YOU!

We know new things can be intimidating and we're here to answer any menstrual cup questions you may have. Seriously, any.

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www.lunette.com
info@lunette.com | @LunetteCup
+358 50 355 4064

APPENDIX

We updated the content of this instructions page on February 28th, 2022. The English version for Lunette Menstrual Cup instructions is the primary and official instruction for use (IFU) and localized language versions are added translations for your convenience.

Please note that local menstrual cup use time recommendations and restrictions may apply, e.g., in France advised, by ANSES 6-8 hours and no overnight use, in Australia, by TGA the maximum use time 8 hours and in Germany 8 hours.

Please note we are not healthcare professionals, and information provided is not intended to substitute professional medical advice and should not be relied upon as such. Please always consult your physician if you have any gynecological issues or medical concerns, such as IUD, irritations, TSS (Toxic Shock Syndrome) or other severe or urgent medical problems. Lunette is always happy to offer troubleshooting tips and advice for your Lunette Cup.

www.lunette.com
info@lunette.fi | @LunetteCup
+358 50 355 4064